



# iCODE CONNECT: ACCESSING YOUR iCODE ON LUNA II PAP DEVICES

Your Luna II PAP device is equipped with many ways of gathering your compliance and usage data. One proprietary way your device does this is by generating a series of 16-digit codes. These codes, called iCodes, hold Summary Therapy data. This data is used to create compliance reports on your device usage. These reports are used by your DME and clinical team to help determine your therapy compliance and effectiveness for your clinical team. This form will walk you through finding the iCodes needed to run these reports.

1

From the home screen (with the device off and not blowing air)



2

Your screen will open to a QR code



3

Twist the knob clockwise, you will see the following iCode Options: iCode1, iCode7, iCode30, iCode60, iCode90 and iCode182. Those represent the last 24 hours, last week, last month, last 2 months, last 3 months, and last 6 months of device usage.



4

You will now see the 16-digit iCode displayed.



5

Scroll to "Back" and click the knob to exit that screen.

